

Sending Data between FORUM and CALLISTO Eye

FORUM to CALLISTO Eye

FORUM

1. Right-click the IOLMaster Report (that also has a REF Image) of the patient and select **Release IOLMaster report**
2. A Worklist entry (Automatic Order) is then required for the CALLISTO Eye.
 - a. Highlight your patient
 - b. Right click
 - c. Select **Create Automatic Order for > CALLISTO Eye**



CALLISTO Eye

1. Select **Patient** (left panel)
2. Click **Import Patient**
3. Click **Planned patients (DICOM)**
4. This should indicate that the patient data you have just sent has been transferred and you will see that the patient has appeared in the patient list

CALLISO Eye to FORUM

1. Go to **Patient Administration**
2. Press the **Export** button
 - i. The "Export" tab is displayed.
 - ii. All treatments satisfying the current filter criteria are listed in the "Treatments" table.
3. Select in the list all treatments to export..
 - i. The storage required is displayed in the "Memory required" field
4. Press the **Export** button
 - i. A window to select the export target opens.
5. Select **DICOM**.

