

IOLMaster to FORUM Data Transfer

FORUM Quick Reference Guide



CAUTION: Refer to the User Manual for warnings, safety information, labeling, and detailed operating instructions. This Reference Guide is only a complement to the User Manual.

- 1 On the IOLMaster®, go to the **Patient** drop down menu and click **Query Waiting Room**. (Figure 1)
- 2 The first patient of the day will open the Worklist. Click on the **Broad Query** tab. (Figure 2)
- 3 Enter today's date. Under **Modality** enter OAM and click **Execute**. (Figure 3) IOLMaster will remember the tab selection, OAM and date until the unit is turned off.
- 4 Highlight the patient from the list and click **OK**.
- 5 Proceed with exam.
- 6 Click the computer icon to transmit the report to FORUM®. (Figure 4)
This is a multiple page report. Use the center arrows in FORUM to scroll through the report.

For prior tests, perform Steps 4 - 6 only.

Figure 1. Patient Drop Down Menu

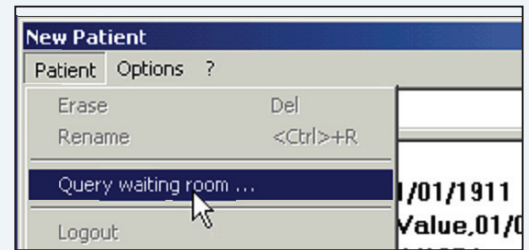


Figure 2. Worklist

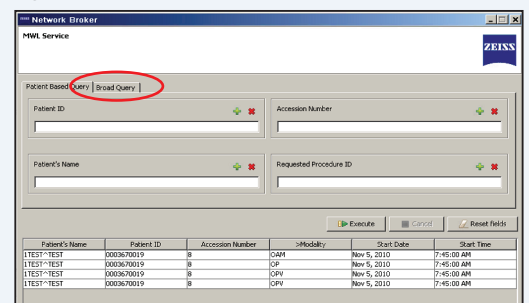


Figure 3. Modality Display Screen


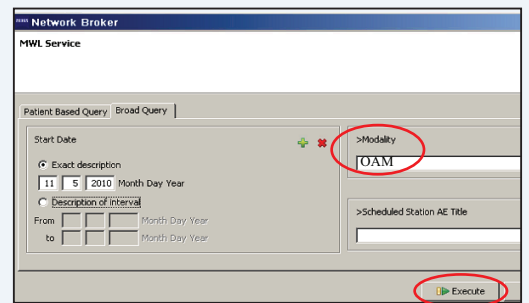


Figure 4. Transmit Report

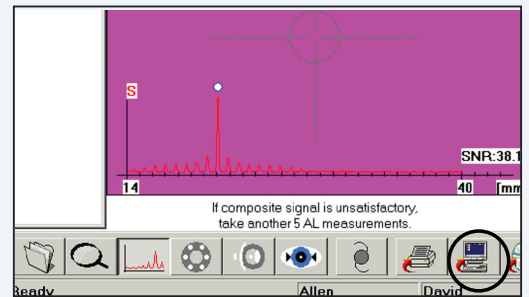

ADDITIONAL RESOURCES: www.meditec.zeiss.com/forum

If you need further assistance, please contact Customer Service at 1-800-341-6968 (USA) or your local Carl Zeiss Representative.

Carl Zeiss Meditec, Inc.
5160 Hacienda Drive
Dublin, CA 94568
USA

Phone: +1 925 557 4100
Toll free: 1 800 342 9821
Fax: +1 925 557 4101
info@meditec.zeiss.com
www.meditec.zeiss.com/us

Carl Zeiss Meditec AG
Goeschwitzerstraße 51-52
07745 Jena
Germany

